



Kirby Lester KL1Plus


The KL1Plus can go far beyond just filling prescriptions. Now, it can help your pharmacy manage controlled medication inventory and eliminate manual logs. (Even use this function for high-value, non-controlled medications). The KL1Plus Inventory Management function provides complete transparency on the medications that need the highest oversight.



KL1Plus Inventory Management Highlights

- Know at all times what is in-stock, and what has been dispensed for patients – from the wholesaler bin, to will call to return-to-stock.
- Perpetual inventory function alerts management of potential discrepancies in real-time.
- Automated C-II log gives pharmacy management real-time access to inventory levels and dispensing history (via web browser).
- Permission-based system tracks activity by user and time via fingerprint scanner.
- Audit inventory discrepancies.
- Establish electronic logs for better compliance with local regulations, and replace labor-heavy manual logs.

Maintain Your Perfect Baseline

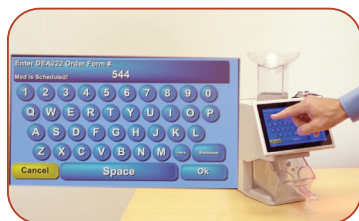


The first step in inventory management on the KL1Plus is capturing new stock as it arrives from the wholesaler, including full information like DEA 222 number and invoice.

Fill Controlled Med Scripts




The KL1Plus will be used to fill scripts for controlled medication. Including scan-verification, double-counting, and stock bottle back-counting. All fill transactions are recorded and available from the web browser application.

Pharmacies fill controlled meds first, then audit inventory

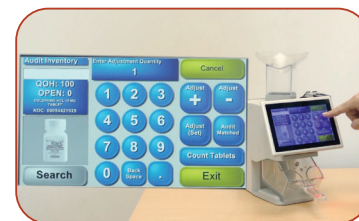

The KL1Plus allows inventory adjustments to be done through the Audit Function (e.g., for periodic audits, broken tablet, manufacturer shortage).

Return-To-Stock Shores Up Inventory Levels

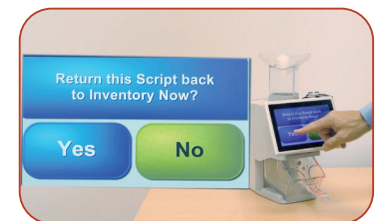

Return-to-stock is also conducted on the KL1Plus. Inventory is updated and recorded after each transaction.